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DE

Lipo Nit® Augenspray - liposomales Augenspray

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Soja-Lecithin, Natriumchlorid, Ethanol, Vitamin A - Palmitat, Vitamin E, Aqua purificata

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Areas of application:

Lipo Nit® Eye Spray stabilises the lipid layer at the lacrimal film, regulates and improves the moisturisation of the eye surface and the eyelids. It should be used in environment-related disorders such as dry eyes, strained eyelids, foreign body sensation, and a burning or itching sensation in the eyes.

Contraindications and hypersensitivity reactions:

Lipo Nit® Eye Spray should not be used in the presence of hypersensitivity to any of the ingredients. Potential hypersensitivity reactions may appear, in very rare cases, in the form of a temporary irritation. The (skin) irritation (e.g. reddening, swelling) is confined to the application area and may be accompanied by a burning sensation or by itching, for example. It appears acutely, is mild in form and subsides rapidly and with no lasting effect following avoidance of the preparation.

Directions for use:

Spray Lipo Nit® Eye Spray 1 - 2 times on the closed eyelids. Do not spray the product into the open eye or touch the eye surface with the spray head. Keep a distance of approx. 10 cm. When using eye make-up, Lipo Nit® Eye Spray should be used before applying make-up and after make-up removal. In principle however, Lipo Nit® Eye Spray can also be used with previously applied eye make-up. In this case, only one squirt should be applied and the distance between the dispenser and the eye should be slightly greater (about 20 cm).

Lipo Nit® Eye Spray suitable for use with contact lenses. Can be used with contact lenses.

General recommendation: use Lipo Nit® Eye Spray 3 to 4 times per day. For more severe cases, Lipo Nit® Eye Spray may be used more often.

Notes:

The bottle may be used by several persons, since there is no contamination.

Lipo Nit® Eye Spray should not be used simultaneously with eye medicine. Should your doctor prescribe you eye drugs other than Lipo Nit® Eye Spray, you should observe an interval of 10 minutes between the two applications.

It is important to check that the seal of the spray pump fits correctly and is undamaged before use.

The bottle is filled generously, because for technical reasons it cannot be completely emptied. It is guaranteed that the indicated quantity can be used.

Packaging contents, appearance and fragrance:

Due to its liposomal formulation, Lipo Nit® Eye Spray is an opalescent liquid with a milky appearance and the characteristic natural fragrance of lecithin.

A brown-coloured opacity or residue may develop during storage, particularly in the area of the spring inside the pump unit. This is a characteristic of the vegetable component (soya lecithin), which has no effect on the effectiveness or tolerance, however.

Shelf-life and storage:

Store in a cool and dry place.

Keep out of reach of children.

Can be used up to 6 months after first use.

Do not use after expiry date.

When using for the first time, you may have to pump 3 or 4 times to fill the tube.

Date of last revision: 2019-01

Symbol/Symbol/Simbolo/Symbol:

Symbol	Hersteller	Fabriqué par	Prodotto da	Manufacturer
	Verfalldatum (JJJJ-MM: Jahr-Monat)	Date de péremption (AAAA-MM: Année-Mois)	Data di scadenza (AAAA-MM: Anno-Mese)	Expiry date (YYYY-MM: Year-Month)
	LOT-Nummer	Numéro de lot	Lotto numero	LOT-number
	Lagertemperatur 2°C - 25°C	Température de stockage 2°C - 25°C	Temperatura di conservazione 2°C - 25°C	Storage temperature 2°C - 25°C
	Gebrauchsinformation beachten	Respectez la notice d'utilisation	Leggere attentamente il foglio illustrativo	Observe instruction leaflet
	Nicht verwenden wenn die Flasche beschädigt ist	Ne pas utiliser si le flacon es endommagé	Non usare se la bottiglia è danneggiato	Do not use if the bottle is damaged
	Nach erstem Gebrauch 6 Monate verwendbar	A utiliser dans les 6 mois après la première utilisation	Può essere usato fino a 6 mesi dopo il Primo utilizzo	Can be used up to 6 months after first use